

Seat Adjustments

It is important to adjust the seat into a good position that allows you to comfortably operate all the vehicle controls.

Forward/Backwards

To move the seat forwards or backwards, grasp the black lever on the right-hand side underneath the seat whilst sitting down.

Push firmly against the floor with your feet.

TOP TIP – Do not use the control tiller to push or pull yourself up.

Backrest Adjustment

The pitch of the seat can be changed by the lifting lever located on the right-hand side of the seat when seated.

Swivel Adjustment

To enable easy access, the seat can be swivelled on a 90 degrees angle.

To change the angle of the seat, sit on the seat, pull the swivel handle towards the front of the scooter and turn the seat anti-clockwise.

